

Author: JTi
Date: 4/2021

EasyOne Connect PDF Export


This Application Note describes the PDF Export capabilities of EasyOne Connect. PDF exports are often used to be able to upload patient test results to an Electronic Medical Record (EMR).

1.0 System Requirements


- EasyOne Connect Version 3.X.X.X

1.1 How to Export a PDF

1. Launch EasyOne Connect.
2. On the Main Screen, Select “Patients”

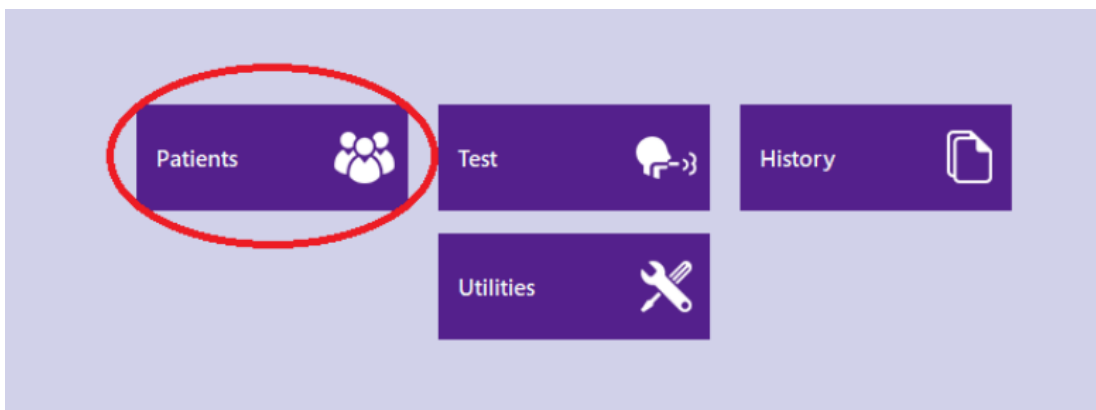

3. Select a patient from the list.

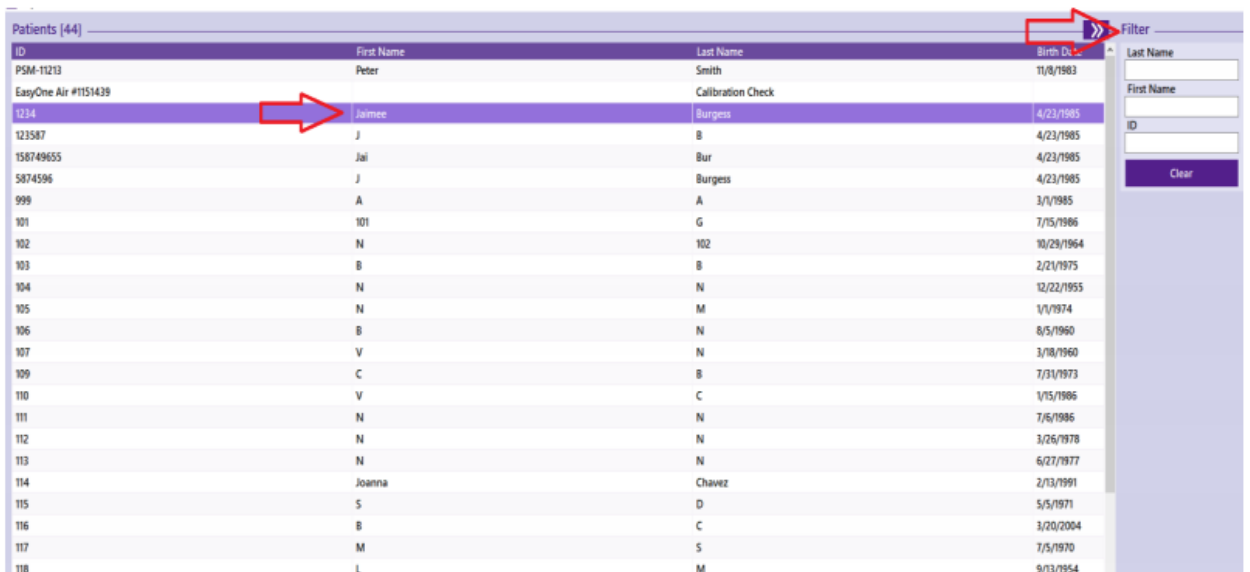

The screenshot shows a table of patients with columns for ID, First Name, Last Name, and Birth Date. The patient 'Jaimee' is highlighted in blue, and a red arrow points to her name. A filter sidebar is visible on the right side of the table.

ID	First Name	Last Name	Birth Dt.
PSM-11213	Peter	Smith	11/8/1983
EasyOne Air #1151439		Calibration Check	
1234	Jaimee	Burgess	4/23/1985
123587	J	B	4/23/1985
158749655	Jai	Bur	4/23/1985
5874596	J	Burgess	4/23/1985
999	A	A	3/1/1985
101	101	G	7/15/1986
102	N	102	10/28/1964
103	B	B	2/21/1975
104	N	N	12/22/1955
105	N	M	1/1/1974
106	B	N	8/5/1960
107	V	N	3/18/1960
109	C	B	7/31/1973
110	V	C	1/15/1986
111	N	N	7/6/1986
112	N	N	3/26/1978
113	N	N	6/27/1977
114	Joanna	Chavez	2/13/1991
115	S	D	5/5/1971
116	B	C	3/20/2004
117	M	S	7/5/1970
118	L	M	9/13/1954

- Select the test you want to print from the test history and click "Report"

Protocol	Test Type	Date	Comment
Base Test	DLCO	1/8/2019 3:40 PM	
Base Test	MBW	1/8/2019 3:11 PM	
Base Test	MVV	1/8/2019 2:11 PM	
Pre / Post	FVL (ex/in)	1/8/2019 2:10 PM	
Base Test	SVC	1/8/2019 1:56 PM	
Base Test	FVC (ex only)	1/8/2019 1:54 PM	
Methacholine (USA PI)	Provocation, FVC Tidal	7/11/2013 1:14 PM	Challenge test example


- Use the Print options on the upper right hand corner to export the PDF how you want. **Note: the report screen can also be accessed directly after performing a test from the test results screen.**

Print **Print Menu**


To send directly to a printer

To save as a PDF or send to EMR

- Once you click on "Print Menu", select "PDF" if you want a PDF. Alternatively, you can select "Export To" and choose the file type you want.

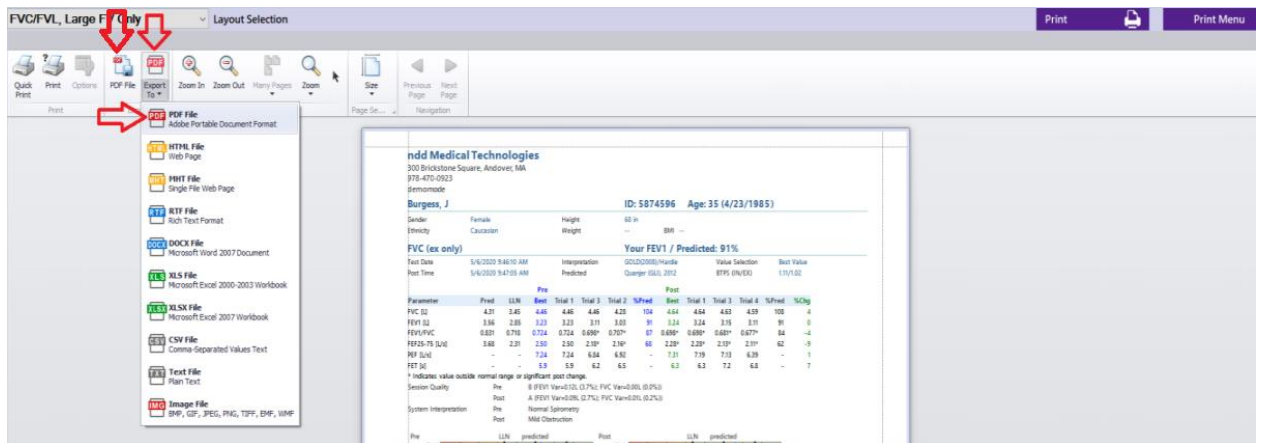

- You can then name and save your PDF file to the desired location.

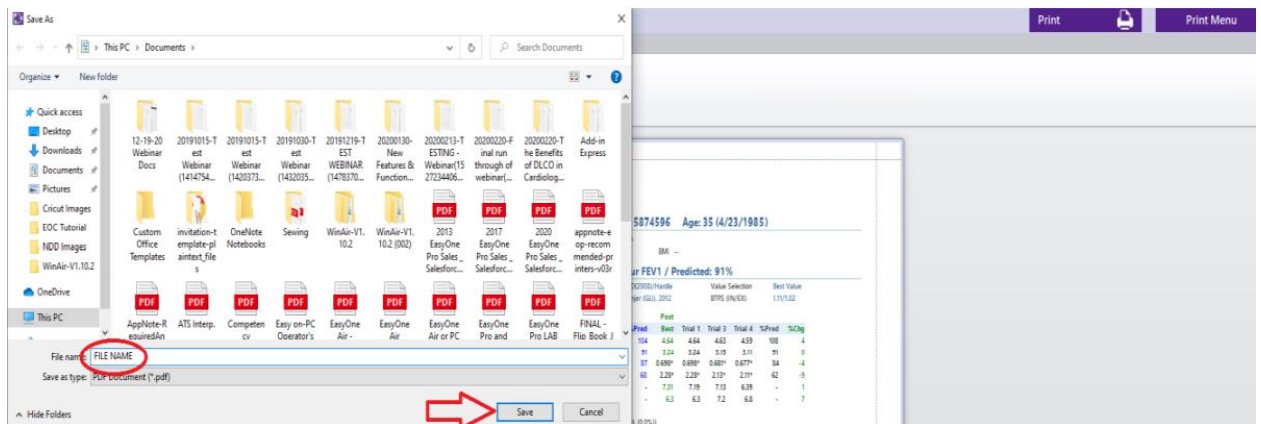

At this point, the saved PDF could be uploaded to your EMR like any other PDF or similar chart attachment.