

Printer Compatibility List

In general, a large number of printers are supported by EasyOne™. Due to the large number of printers available on the market, ndd cannot test every printer for compatibility. EasyOne™ supports three different principal driver types: Hewlett-Packard (HP) PCL (PCL with GUI works, but without guarantee), Epson ESC/P2, and Canon Extended Mode (CEM) commands.

Many laser printer manufactures like Brother, Canon, Kyocera, etc. have PCL compatible models and work with the HP b&w setting.

Matrix Printers: Many matrix printer manufactures like Epson, Panasonic, etc. have ESC/P2 compatible models and work with the EPSON b&w setting. Please check the specifications of the printer for the ESC/P2 mode.



New: All PictBridge printers can be used with the PictBridge cradles (serial number 80'000 and higher are fitted with special PictBridge hardware)

List of tested printers that work using direct connection to EasyOne™ cradle:

Manufacturer	Printer Model (current models highlighted)	Printer Type	EasyOne Setting	Parallel	USB	Note
Apollo	1000	Inkjet	HP b&w	x		
Brother	B4300, MFC8820D, HL5150D, HL5140 HL2070N	Laser	HP b&w	x	x	
Brother	MFC-8860DN, HL2150-N, DCP 7010L	Laser	HP b&w		x	
Brother	MFC-9420CN, MFC-9840CN	Laser	HP b&w / color		x	
Canon	BJC-620, 2000, 4100, 7000, 7100	Inkjet	Canon b&w / color	x		
Canon	BJC-85, 2100, 2110	Inkjet	Canon b&w / color	x	x	
Canon	S400, S450	Inkjet	Canon b&w / color	x	x	1
Canon	iP2000, (iP2200 does NOT work), iP4200, iP4300, iP5200, iP90 (portable), iP6100, i80 (portable)	Inkjet	Canon b&w / color		x	7
Canon (Jap)	BJ-M40	Inkjet	Canon b&w / color	x	x	5
Epson	Stylus Color 2, 600, 680, 740, 900	Inkjet	Epson b&w / color	x		
Epson	C60, C62 (C64 does not work)	Inkjet	Epson b&w / color	x	x	
Epson	C70, C80	Inkjet	Epson b&w	x	x	3
Epson	Stylus Photo	Inkjet	Epson b&w / color	x		
Epson	Stylus Photo 750	Inkjet	Epson b&w / color	x	x	
Epson	LQ 570	Matrix	Epson b&w	x		2
Epson (Jap)	PM780C, PM800C, PM880C, PM950C, PM2200C, PM3000C, PM-G800, PXV700, PX- G900	Inkjet	Epson b&w / color		x	5
HP	LaserJet 4L, 4M, 4MP, 6L, 4050, 4250	Laser	HP b&w	x		
HP	LaserJet 1220, 1200 series, 1150, 1300, 2300 series, P2015, 2055, P3015	Laser	HP b&w		x	4
HP	OfficeJet G85	Inkjet	HP b&w / color	x	x	6
HP	OfficeJet 6000, OfficeJet 8000 (OfficeJet 8000 Enterprise does not work) OfficeJet 6100, OfficeJet 8100Pro, OfficeJet 100, HP Officejet Pro 6230 eprinter	Inkjet	HP b&w / color		x	8
HP	Deskjet 350C, Business inkjet 2230, 2280, 2600	Inkjet	HP b&w / color	x		3
HP	Deskjet 340, 350, 500, 510, 520, 540	Inkjet	HP b&w / color	x		3
HP	Deskjet 550C, 610C, 640C, 840C, 850C, 895C, 940C, 948C, 990, 1120C	Inkjet	HP b&w / color	x		

Manufacturer	Printer Model (current models highlighted)	Printer Type	EasyOne Setting	Parallel	USB	Note
HP	DeskJet 450, 930, 3820, 5160, 5550, 5551, 5560, 5650, 5652, 6122, 7870, 9300	Inkjet	HP b&w / color	x	x	
HP	460CB (portable Firmware >= V2.16!), 470 812C, 842C, 5150, 5151, 5440, 5740, 5940, 6540, D5069, 6940, 6988	Inkjet	HP b&w / color		x	
HP	D5160, D6160, D5360, D5460	Inkjet	HP b&w / color		x	8
Kyocera	FS-1118MFP	Laser	HP b&w / color	x	x	
Lexmark	E250D	Laser	HP b&w		x	
Nashuatec	C7417n (with HP b&w setting)	Laser	HP b&w		x	3
Oki	8Pplus, B4350	Laser	HP b&w	x	x	
Samsung	ML-7000	Laser	HP b&w	x		
Xerox	P8ex	Laser	HP b&w		x	3

Notes:

- 1 If EasyOne firmware version is 1.15 or lower, the page size is reduced to 1/4 page.
- 2 Matrix printers in general are not recommended since the printout time is very high. If the resolution is set too low (e.g. 180 dpi), curve data may be missing on printout.
- 3 Works only in b&w mode.
- 4 Works only with USB cradle firmware > V01.00.00.00.
- 5 Japanese models.
- 6 Must be set to high resolution, otherwise curve is missing.
- 7 If EasyOne firmware version is 2.14 or lower, the page size is reduced to 1/8 page.
- 8 Works only with Firmware >= V02.18.00.00 , Firmware >= V03.17.00.00 or Firmware >= V05.00.00.00.

Additional Notes:

- All printers are supported if the printing is done 'via PC' using the EasyWare PC software.
- Many low cost laser printers use the GDI language which is not supported by EasyOne™.
- Some matrix printers are supported but the printout is extremely slow.