

EasyOne Pro Install Printer Driver (Firmware \geq V 1.3.0.3)



Preparation

- Download printer update file from the ndd Homepage to a memory stick:
<http://www.ndd.ch/Downloads/software.aspx>
- Disconnect all printer from EasyOne Pro
- Connect USB stick to the EasyOne Pro


Update Procedure

- Utilities \rightarrow Advanced \rightarrow Enter Password: EOPTM \rightarrow Update Software \rightarrow browse memory stick \rightarrow open the update folder (e.g. HP_Dekjet_xxxx)

- Run 
EwpUpdater.exe

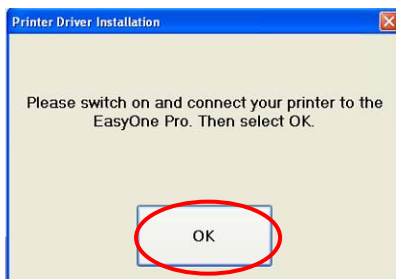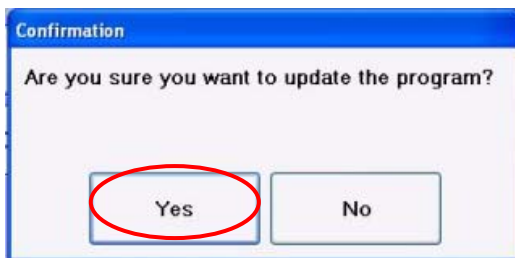

- The printer driver will be installed...

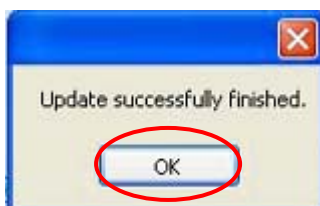

- Wait until the EasyWare Pro software has started
- Restart the EasyOne Pro

Select Printer as default Printer

- Utilities \rightarrow Configuration \rightarrow Printer.
- Select default printer from the list "Direct Print to"
- Press OK.